

C82 Aura Transmitter for Hearing Aids



Interactive Setup Guide

Version 1.5

Start Setup

[< Back](#)


Main Menu

1. [Before You Start](#)
2. [Buttons & Indicators](#)
3. [Compatible Hearing Aids](#)
4. [Use With App](#)
5. [Use with Samsung Phone](#)
6. [Advanced Features](#)
7. [Register Your Product](#)
8. [Live Agent Help](#)
9. [Rate This Guide](#)
10. [Advocate for Us](#)

[Next](#)

< Back

Back to
Menu

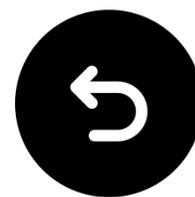


1. Before You Start

- Auracast-only dongle
(not for standard Bluetooth)
- Sends audio to Auracast devices
- Supports hearing aids
→ [[See Supported Devices](#)]
- Default mode: Hearing Aid

Next



2. Buttons & Indicators

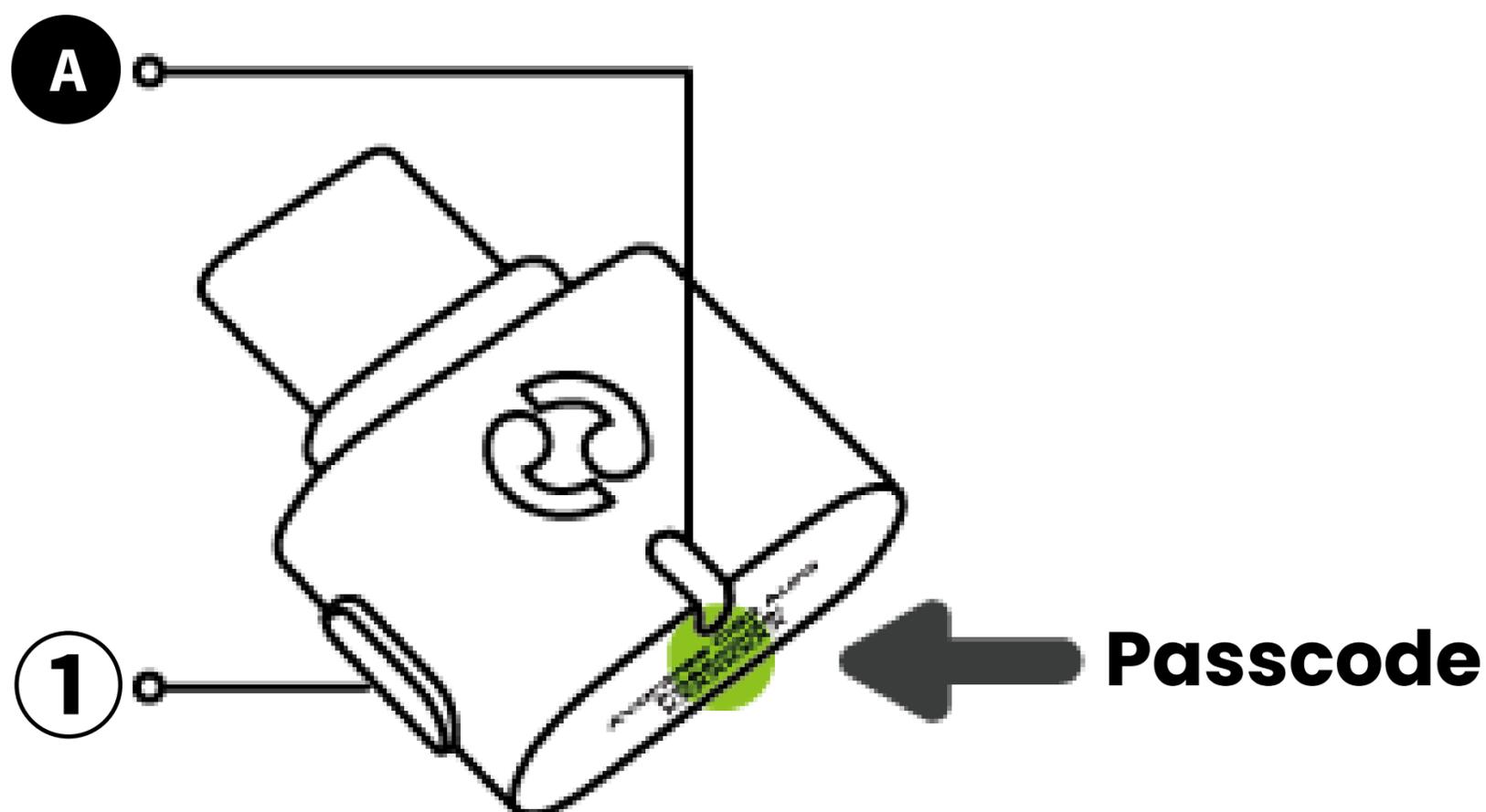

- Press and hold ① MFB for 6s to switch mode: → LED ● A :
 - **White** = Hearing Aid
 - **Orange** = HD
- LED solid = Broadcasting



3. Compatible Devices

3.1 Auracast-Enabled

Connect via your hearing aid's companion [app](#), or an Auracast

[Assistant \(e.g. Samsung S24:](#)

- ReSound Nexia,
- ReSound Viviva
- ReSound Savi
- Beltone Commence
- Beltone Envision
- Beltone Serene
- GN Danalogic Extend
- Jabra Enhance Pro 20
- Jabra Enhance Pro 30
- ...

< Back

Back to
Menu



3. Compatible Devices

3.2 Auracast-Ready

Requires [an Auracast Assistant \(e.g. Samsung S24\)](#) to connect:

- Starkey Edge AI
- ...

Note:

Device support is updated regularly. Check [FAQ](#) for the latest list.

Next

< Back

Back to
Menu



3. Compatible Devices

3.3 Choose How to Use




With App



With Samsung Phone

< Back

Back to
Menu

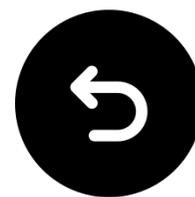


4. Use With App

1☒ Plug in C82 Aura and play audio.

2☒ On your hearing aid's app: go to **LE Audio/Auracast**.

3☒ Tap **Aura_XXXX**, then enter the password. → You're Ready.

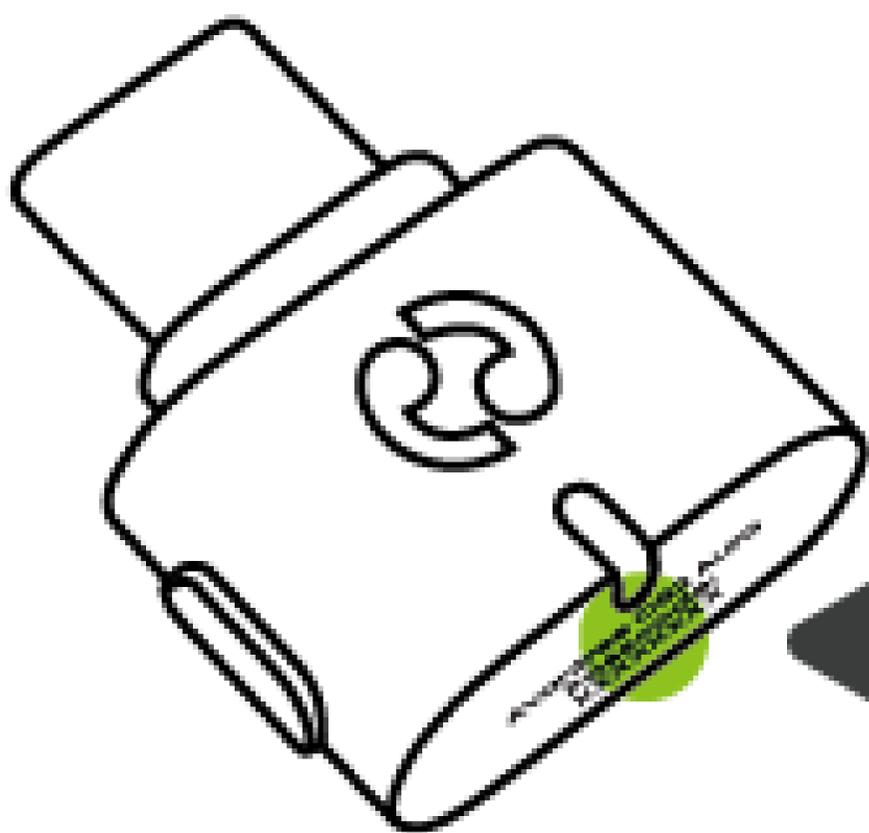


Passcode

Next

< Back

Back to
Menu

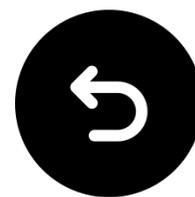

5. Use With Samsung Phone


1☒ Plug in C82 Aura and play audio.

2☒ Turn on Bluetooth and choose your hearing aids.

3☒ Tap  and turn on **LE Audio**.

4☒ Tap **Listen to Auracast broadcast**
> **Aura_XXXX**.

5☒ Enter the password. → You're Ready.

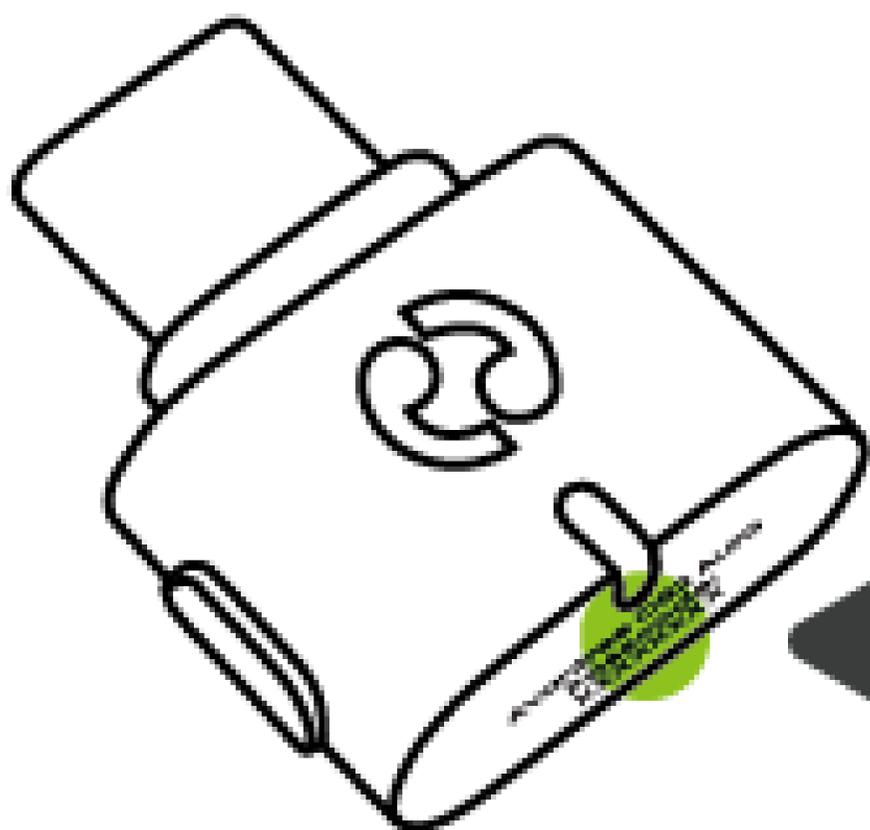


Passcode

[Find Samsung Phone Alternatives](#) ↗

Next

< Back

Back to
Menu

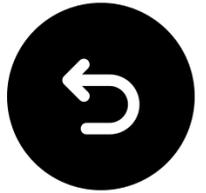

6. Advanced Features

6.1 Firmware Update

1☒ Download [Avantree Updater](#) on your PC.

2☒ Plug in C82 Aura.


3☒ Install and run the app.

4☒ Follow the in-app steps.

Next

< Back

Back to
Menu

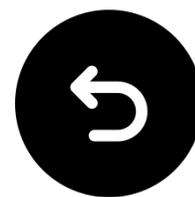

6. Advanced Features

6.2 HD Mode Setup


C82 Aura supports HD sound with Auracast headphones like [Avantree Infinity](#).

[Contact us](#) for how to set up.

Next

< Back

Back to
Menu

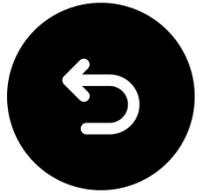

7. Register Your Product

Register now to:


- Get notified of [firmware updates](#).
- Extend warranty to 24 months—
FREE.
- Enjoy priority service support.

[Register Now](#) ↗

You're Ready

< Back

Back to
Menu

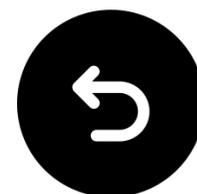

8. Live Agent Help

We'll respond within **1 business day**

Quick support options:

☒ Email: support@avantree.com

☒ [Schedule callback ↗](#)

☒ [Submit a ticket ↗](#)

☒ [Live Chat ↗](#)

< Back

Back to
Menu




9. Rate This Guide

Thanks for choosing C82 Aura!


We hope this guide helped you get started.

Your quick rating helps us improve:

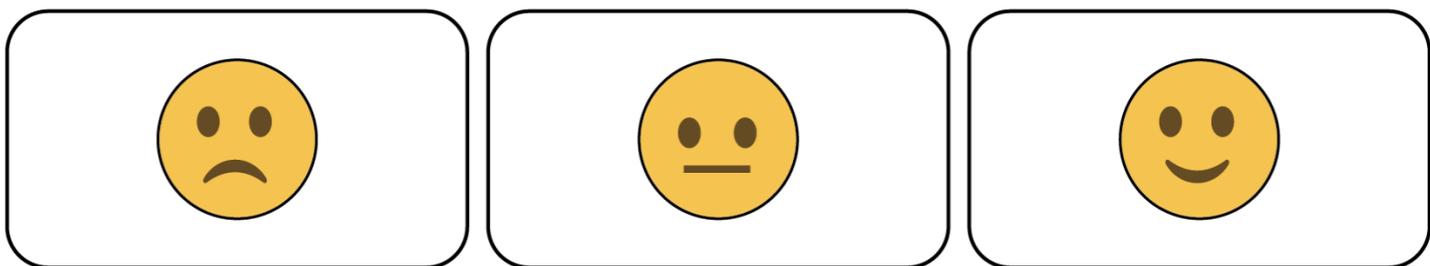

< Back

Back to
Menu

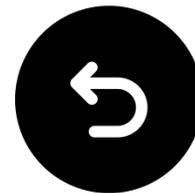

10. Advocate for Us

We've supported hearing-impaired customers for **20+** years.

Did we help you? Leave a quick review on Amazon to support us!

Select your store:

[Amazon US](#)