



DG60 Aura

USB Transmitter for
Auracast Hearing Aids



Interactive User Guide

Version 1.8

Start Setup

< Back



Main Menu

1. Before You Start
2. Buttons & Indicators
3. Connect to PC
4. Connect with Device
 - 4A. Auracast Devices
 - 4B. Bluetooth headphones
5. FAQ & Troubleshooting
6. Support & Feedback

Next


1. Before You Start

DG60 Aura is a USB audio transmitter. It supports **Bluetooth** and **Auracast™** wireless streaming.

1. Works with all Bluetooth headphones/earbuds.
2. Streams to Auracast™ hearing aids & devices. [Compatibility Checker](#) ↗
3. No driver required.
4. Supports **Windows PC & Mac**.

2. Buttons & Indicators

Mode Switch

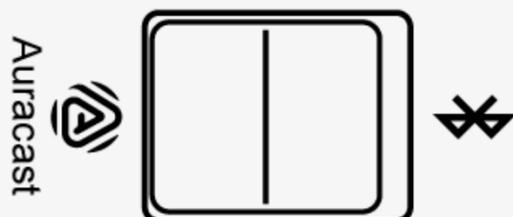

- Slide to  → Bluetooth mode
- Slide to  → Auracast mode

MFB Button

MFB

- **Bluetooth mode:** Hold 3s → Enter pairing →  +  flash
- **Auracast mode:** Double click → Switch ( = Hearing aids /  = Standard)

 [Full Buttons & Indicators Guide](#)

Next

< Back



Menu

3.1 Assemble & Connect

- 1 Attach antenna to dongle.
- 2 Plug into PC.




Next


3.2 Select as Audio Output

1 Open **Sound Settings** on your PC.

- **Windows:** Settings → System → Sound

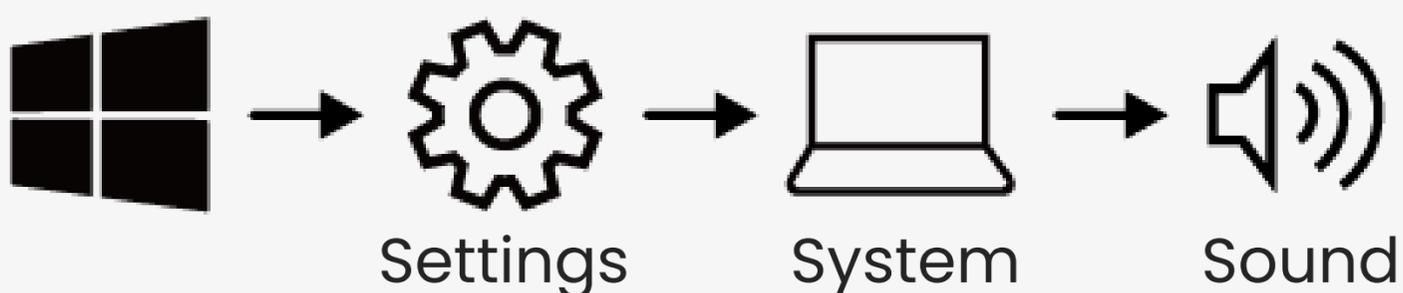

- **Mac:** System Settings → Sound

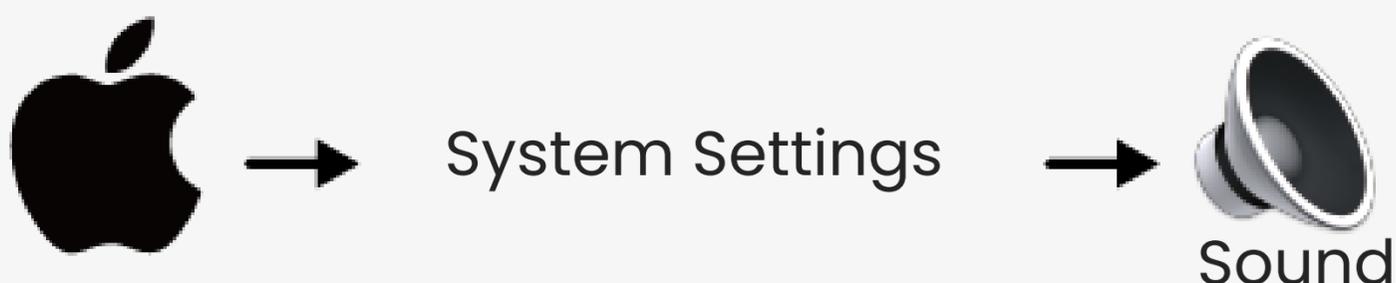

2 Choose Avantree DG60 Aura as **default output**.



4. Connect with Device

Choose how you want to use
DG60 Aura:

A



[Auracast Devices](#)

→ Connect to Auracast-ready
hearing aids or Speakers

B




[Bluetooth Headphones](#)


→ Connect to your Bluetooth
headphones /speakers

4B.1 Turn Off Phone Bluetooth

Turn off phone Bluetooth to avoid auto-reconnect and pairing issue

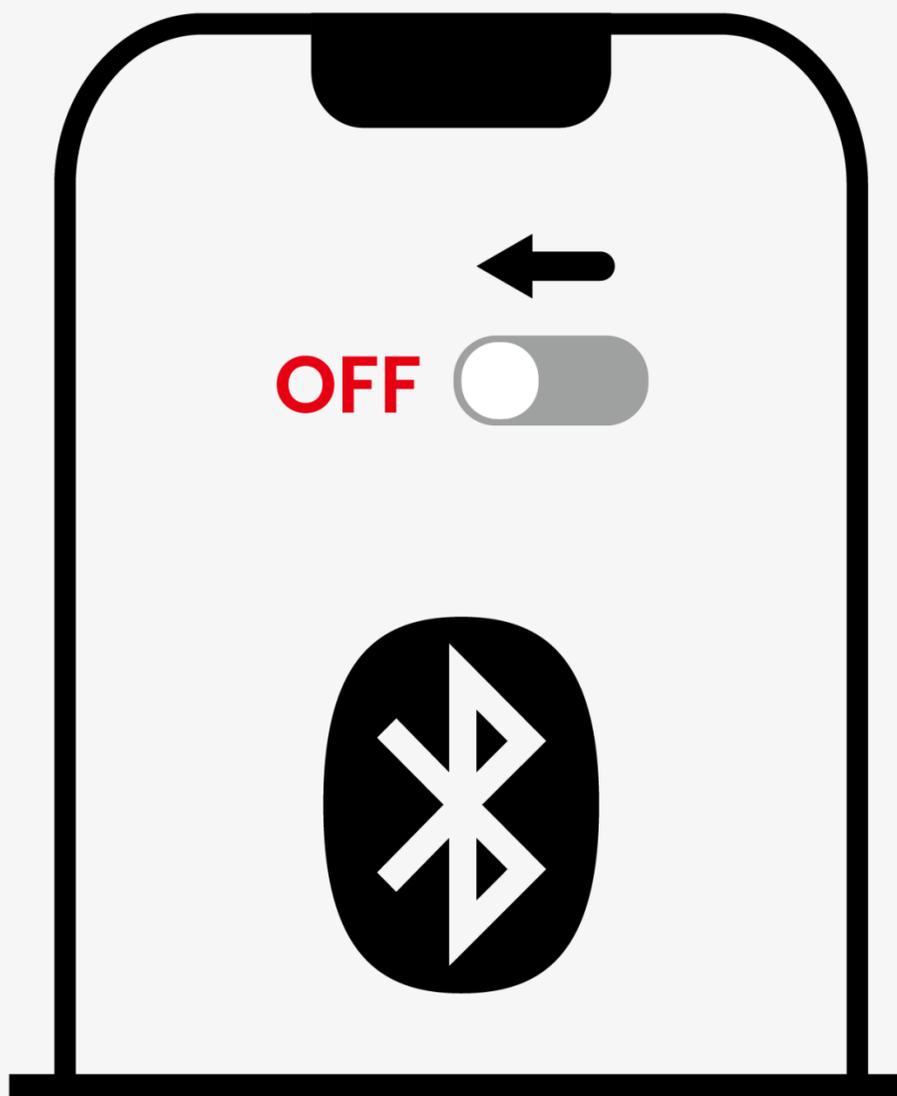

4B.2 Enter Bluetooth Pairing

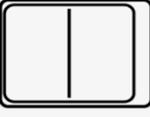- Slide   to 
- LED blinks **blue** & white → Pairing mode.




 If LED not blinking **blue** & white, **press & hold** MFB until it does.

Next

4B.3 Put Headphones in Pairing

- Headphones **must be in pairing mode**.
- Usually, press & hold the main button until the light **blinks**.

Example: AirPods Pro pairing mode

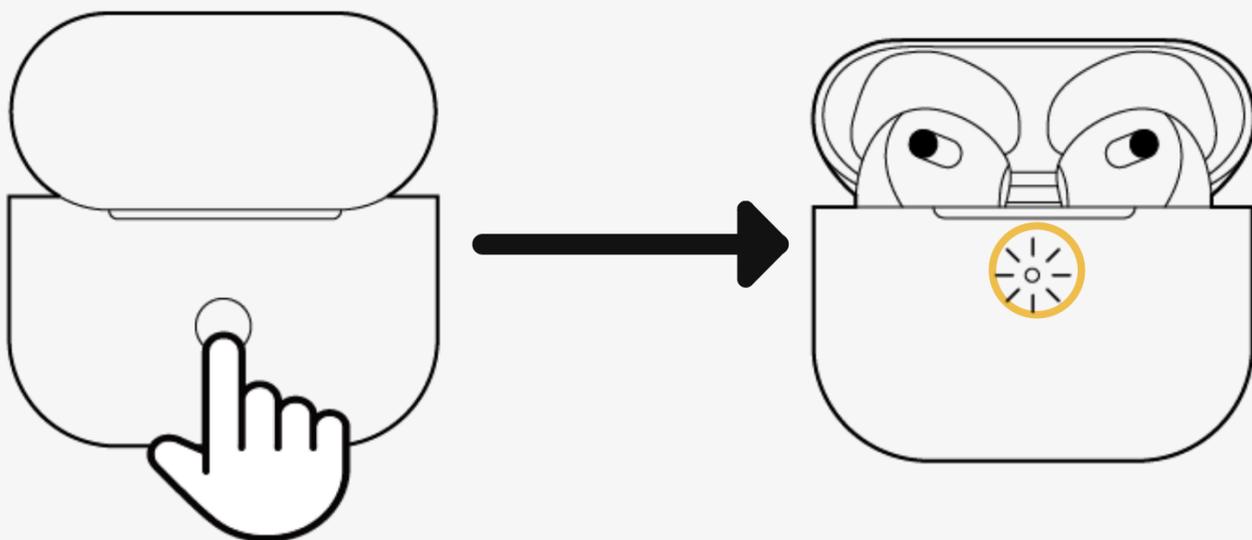

 Most failures happen when headphones are not in pairing mode.

 Need help? Check our

[Headphone Pairing Guide](#)

Ready to Pair

4B.4 Connect & Play

❶ Place headphones near transmitter.

✓ Connected: LED turns **solid blue**

❷ Play Audio → Enjoy audio through headphone.



[!\[\]\(3b71157eab31889e641f7620692f0b92_img.jpg\) Poor Sound ?](#)

Next

4B.5 Talking (Optional)

To use mic, set **Input device** → DG60 Aura

1 On your PC:

- **Windows:** Settings → System → Sound → Input → DG60 Aura
- **Mac:** System Settings → Sound → Input → DG60 Aura

2 In your call app (Zoom / Teams / Skype):

- Select **DG60 Aura** for Mic & Speaker

 In call mode, **LED = White solid**

 [Using with Auracast device?](#)

 Auracast mode does not support mic talking.


4A.1 Auracast Must Know

1. **New Tech** – Only the newest devices support Auracast.
[Check here](#) ↗
2. **Hearing Aids** – "Auracast-ready" ≠ enabled. Only active Auracast works
3. **Connection** – Needs an assistant: device app or Android/Google Auracast Assistant.



Important: Auracast devices need to either use an app/assistant or new Android Auracast Assistant to find broadcast channels

4A.2 Start Broadcasting

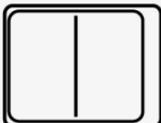1 Slide   → 

2 Double click  → Switch between

- white → Auracast Hearing aids
-  orange → Auracast headphones/speakers




Choose Auracast Assistant


Your device needs an Auracast Assistant to find broadcasts.

- Some use **their own app** (e.g. ReSound).
- Some rely on the **Android Auracast Assistant** (e.g. Starkey).

Now select your setup:

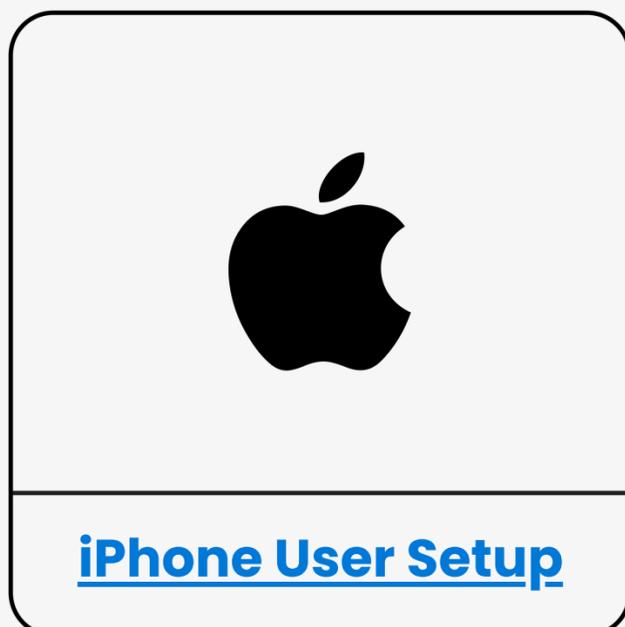


Use your **hearing aid's app**

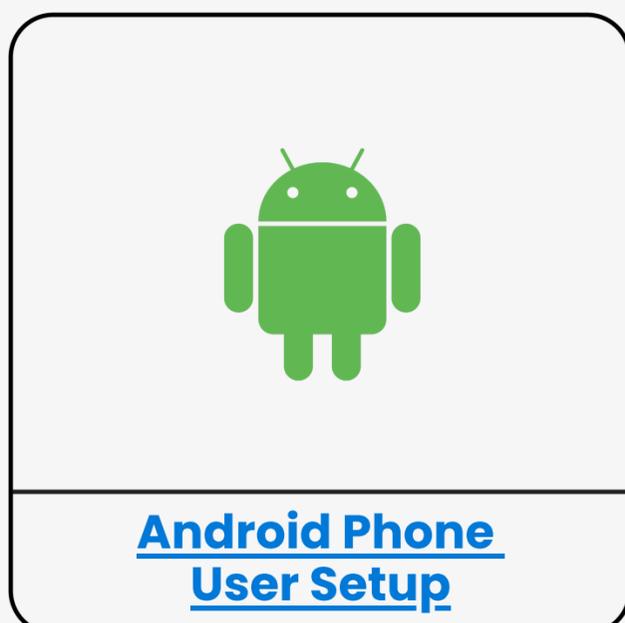

Use your **hearing aid's app** or the **Android Auracast Assistant** (Android 15+)

4A.3a iPhone User Setup

- 1 Pair your device with phone via bluetooth.
- 2 Open device manufacturer's app: Turn on **Auracast assistant** → Find a Broadcast → **DG60 Aura**.
- 3 Play Audio → Enjoy with Auracast device.

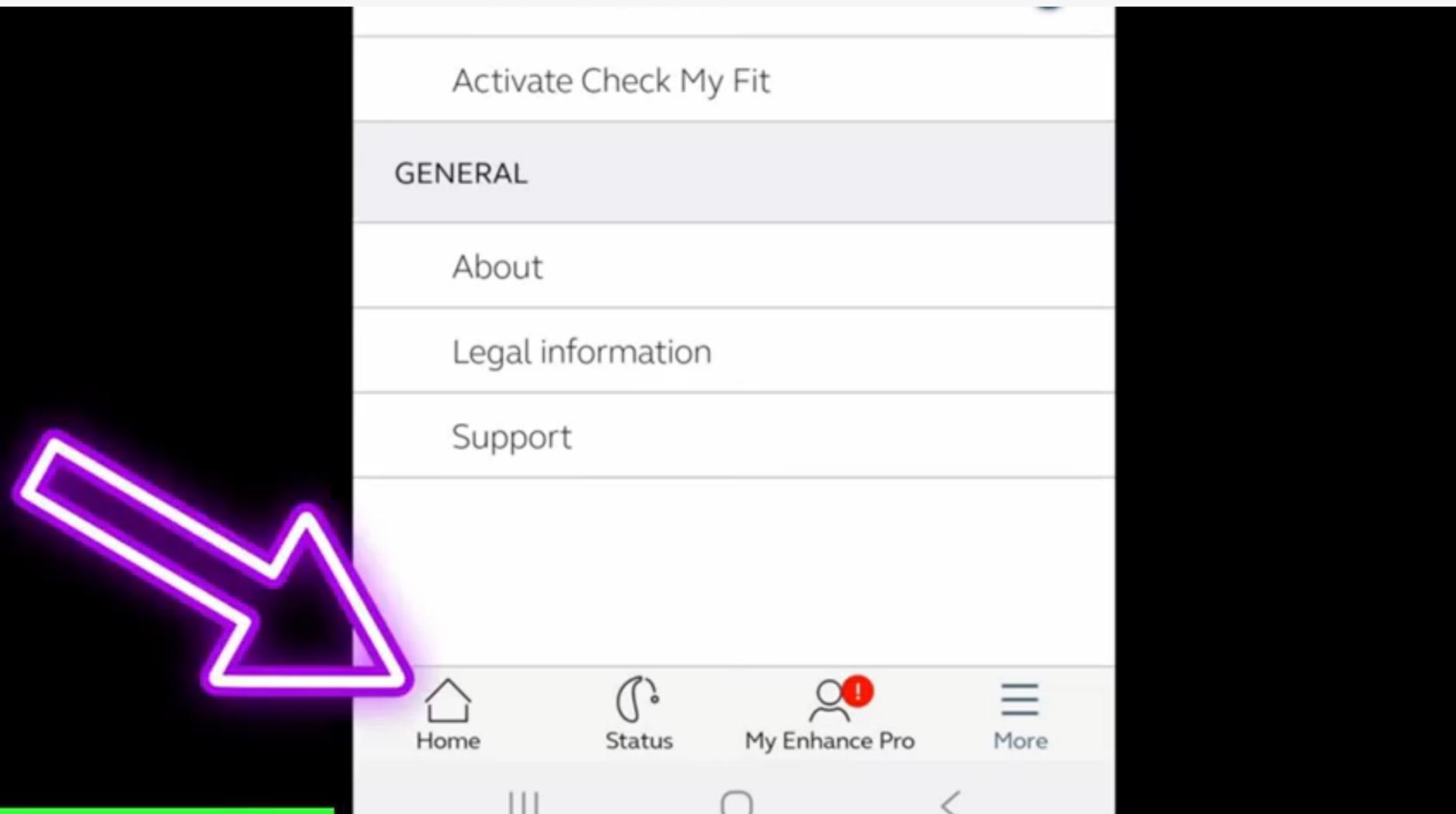


 iOS does not support Auracast natively. You must use your device's App Auracast Assistant.

Next

4A.3b Android Phone User Setup

- 1 Pair your device with phone via bluetooth.
- 2 Turn on LE Audio on Phone
→ tap **Listen to Auracast broadcast**
→ **DG60 Aura**.
- 3 Play Audio → Enjoy with Auracast device.

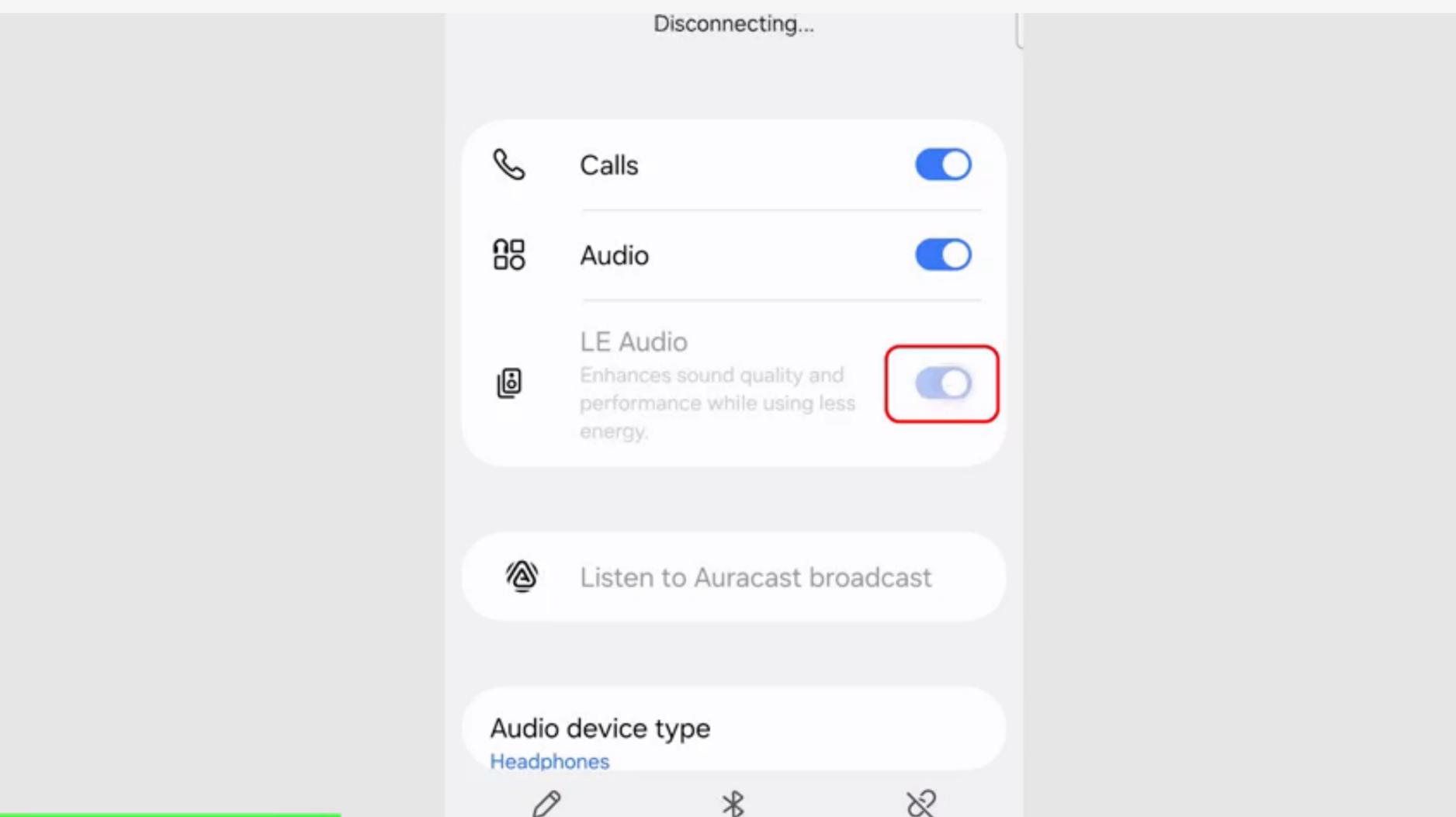

[Find a phone with Auracast Assistant support.](#) (e.g. Samsung S23+, Pixel 8/9)

Next



4A.4 Stable Connection

- Turn **OFF** phone Bluetooth **after** hearing aids connect
- Prevents phone signals from disturbing Auracast.



Mic note

- Auracast mode does not support microphone.
- For talking, [use a Bluetooth headset](#) ↗

5. FAQ & Troubleshooting

Here are some common questions:

- [Won't pair? ↗](#)
- [Poor sound? ↗](#)
- [Mic not working? ↗](#)
- [Will Sony WH-1000XM5/XM6 work? ↗](#)
- [How do I reset /clear pairing history? ↗](#)

 [More FAQ & Troubleshooting ↗](#)

< Back



Menu

7. Support & Feedback

1. Register Your Product

Extended warranty & get updates.

[Register Your Product](#)

2. Agent Support

Need help? Contact our agents.

 Email

 Call


 Chat

3. Share Feedback


How do you feel about this guide?



Bad

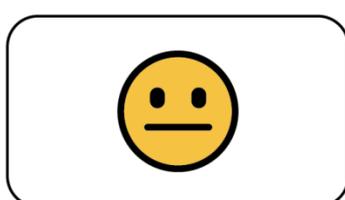

Okay

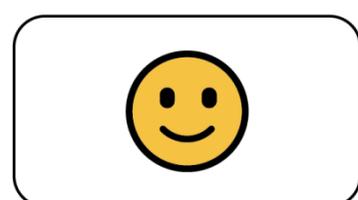

Good