




Voyager HA

Auracast Transmitter for
Hearing Aids

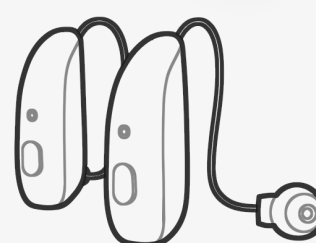

Interactive User Guide

Version 2.4.5



Start Setup



Firmware Update

< Back



Main Menu

1. Before You Start
2. Use with Hearing Aids
3. Connect to Audio Source
4. Adjust Hearing Experience
5. Use with Bluetooth Headphones
6. Firmware Update
7. Buttons & Indicators
8. Advanced Setup
9. Troubleshooting
10. Register & Support



Rate This Guide

Next



1. Before You Start

- **Connection modes**

Auracast – for [Auracast hearing aids](#) ↗

Bluetooth – for Bluetooth headphones **(1 device only)**

- Works with devices that have a **3.5mm audio output**
- **Not recommended for TV use**

Next

2. Use with Hearing Aids

2.1 Start Broadcasting

Works with **Auracast-enabled** hearing aids. See [Compatibility Checker](#) ↗

❶ Slide to 

❷ Screen shows [**Hearing Aids**].

⚠ *If not, double-press* 



< Back



Menu

2.2 Join Auracast with your hearing aid app



You'll open a new guide.
Come back here when finished.

ReSound GN

danalogic GN

oticon

Jabra GN



WIDEX



 **Beltone**™

Bernafon
THE SOUND EXPERIENCE

PHILIPS


Next

Join with Auracast Assistant

- Available on newer Android phones.
[Check supported phones](#) ↗
- **Widex** supported model:
Allure (firmware $\geq$ 5.1.20)

Example: Samsung Galaxy S24

- ➊ Connect hearing aids with phone.
- ➋ Open phone Bluetooth setting.
- ➌ Tap **Listen to Auracast broadcast**.
- ➍ Select a broadcast.

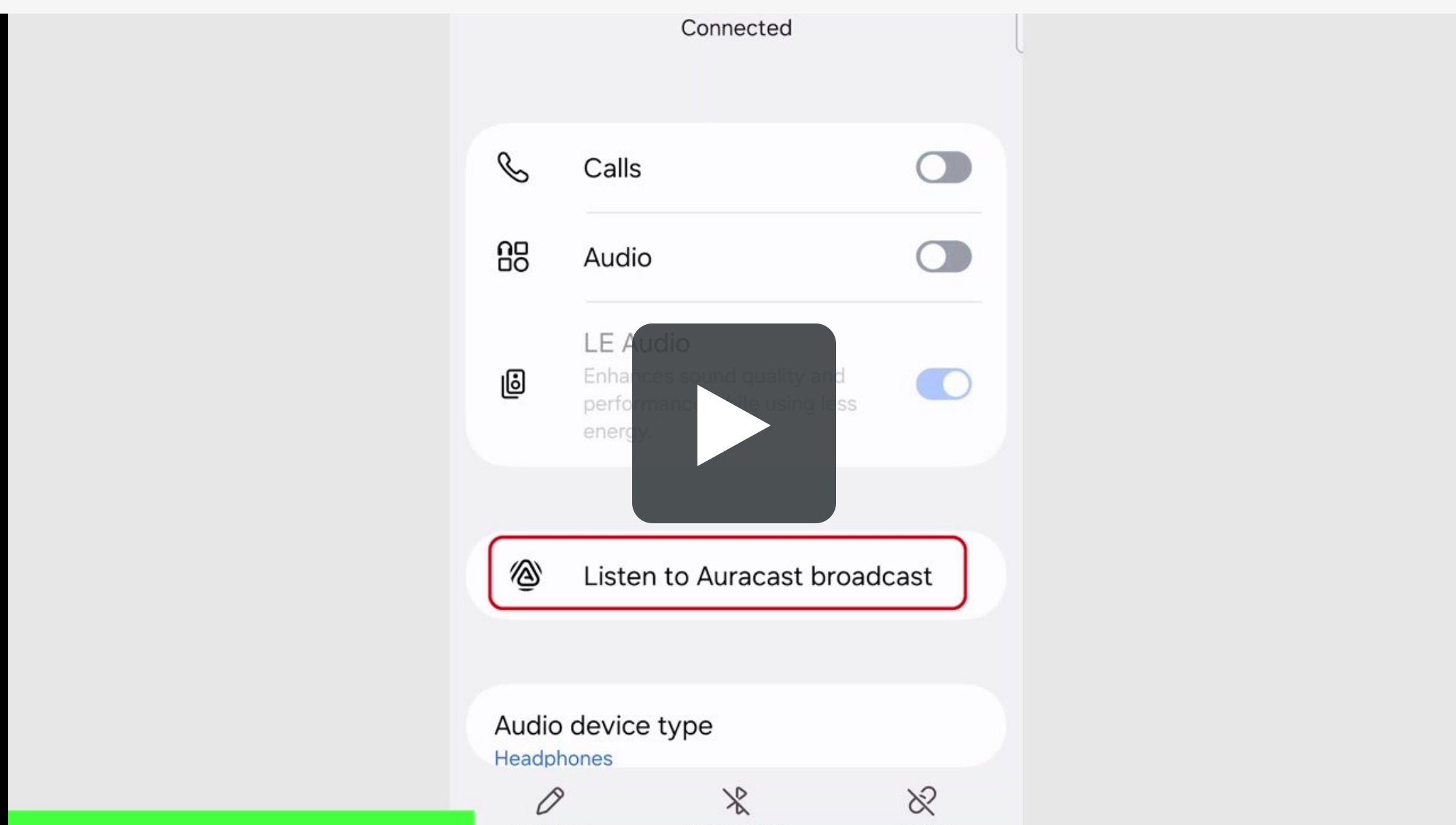[Next](#)


3. Connect to Audio Source

Works with devices that have a 3.5mm audio output

Examples:

- Airplane / in-flight entertainment systems
- Treadmills / fitness equipment
- Laptops, PCs, or other AUX devices

Hear audio wirelessly via your hearing aids

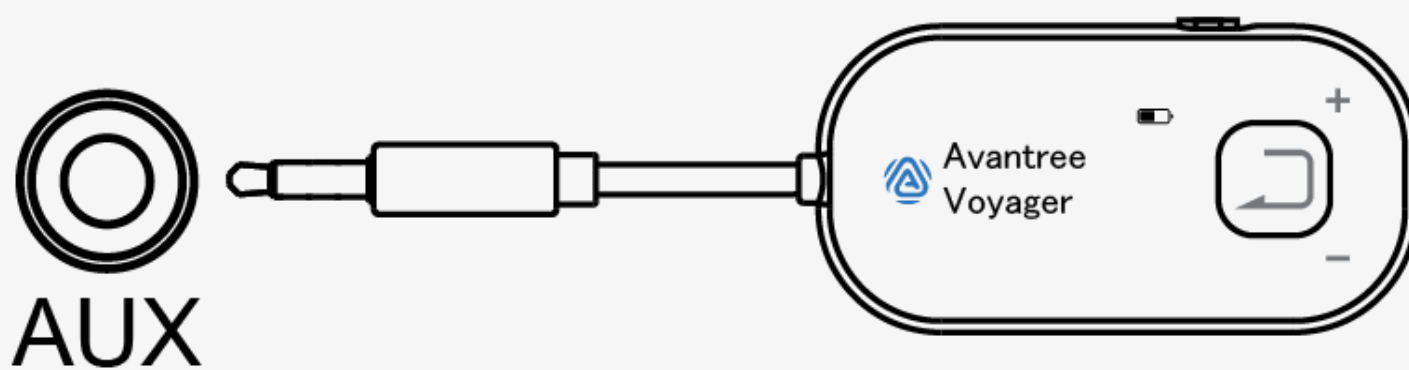


Note: If Auracast audio cuts, turn **OFF** phone Bluetooth **after** hearing aids connect.

4. Adjust Hearing Experience

Control your hearing experience in your **hearing aids app**

Adjust both sound quality and surrounding sound (Mic Mixing if supported)

Example shown using Starkey hearing aids app:

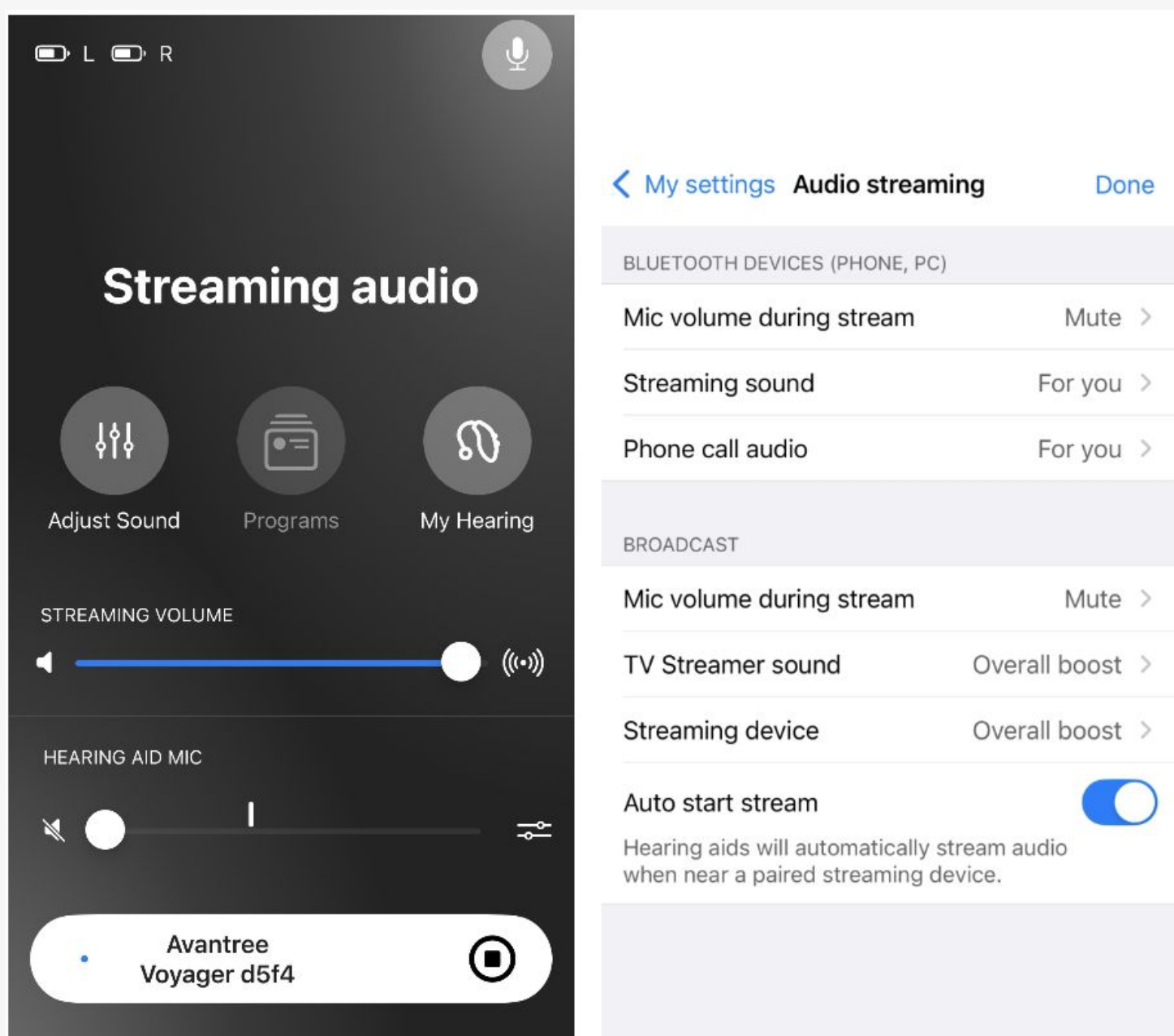


Other brands may use different labels or layout

Next

Adjust Volume

- ❶ Adjust on your hearing aids / app.
- ❷ Use + / - on Voyager HA.
- ❸ Set volume on the source device.

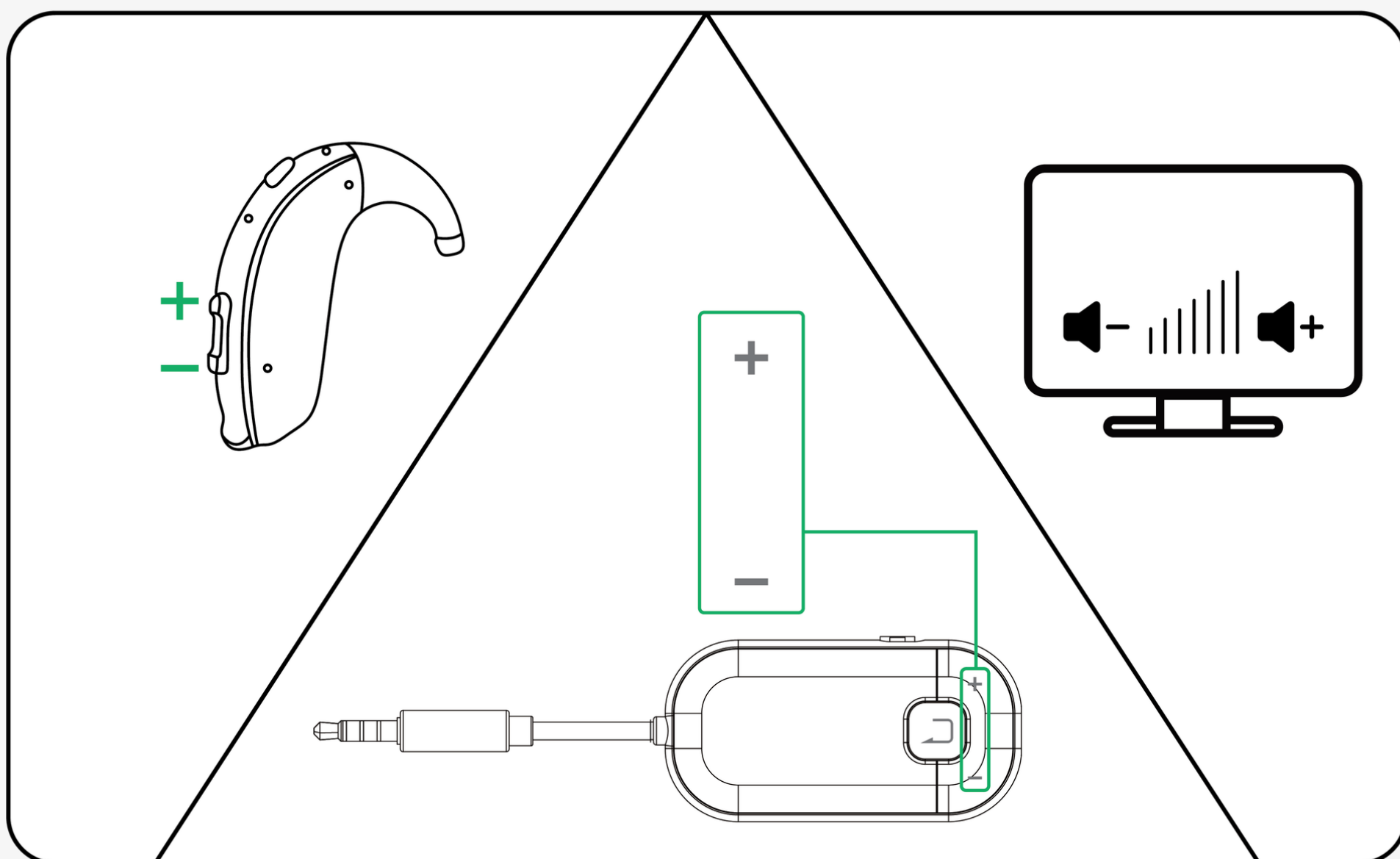



5. Use with Bluetooth Headphone

❶ Slide Voygaer HA to **ON** or  → Screen shows **[Searching]**.

If not, hold  2s.

❷ Ensure headphones in pairing mode

 Not sure? [Headphones Pairing Guide](#) ↗

❸ Use + & - to select your device.

Press  once to connect 

 *The power label may vary by batch: **ON or** *

Next



6. Firmware Update

Keep Your Transmitter Up To Date



Latest Firmware

v20.6.0



Customize the broadcast name and passcode

Takes about 1 minute



Update

→ [View full version history](#)



Get Future Update Alerts

- Compatibility improvements
- New features
- No promotions. Ever.

Notify Me

Next

7. Buttons & Indicators



The label may vary by batch, but both work the same.



Switch modes:

- **Auracast:** 
- **Bluetooth:** **ON** or 



Switch Auracast device type:

- **Hearing aids**
Auracast hearing aids
- **Standard**
Auracast headphones/speakers

[!\[\]\(43fda5baa5446493352974e4b4060607_img.jpg\) Full Buttons & Indicators Guide](#)

Next



8. Advanced Setup

8.1 Customize & Update

Customize broadcast name and passcode

① Upgrade firmware.

- Install [Avantree Firmware Updater](#)
- Update to the latest version

② Customize in Updater App

Once upgraded, open the Updater app → **Go to Setting**

- Customize the broadcast name.
- Set a custom pairing passcode for enhanced security.

Next

8.2 Use with Auracast Headphones

Supports Avantree Auracast headphones and receivers

- 1 Slide Voyager HA to 
- 2 Turn on your headphones/receivers
→ Auto-join the broadcast



AuraGo



AuraClip



AuraLink



Aria Aura



Aria Aura T



Loop (Dock)



9. Troubleshooting

Common Issues

- [Can't connect to hearing aids ↗](#)
- [Hearing Aids Disconnect ↗](#)
- [Compatibility quick check ↗](#)

[👉 More FAQ & Help ↗](#)



Still need help? [Contact support ↗](#)

Next

10. Register & Support

Register Your Product

Extended warranty & get updates.

[Get 24M Warranty](#)

Need help? Contact Us

 Email
support@avantree.com >

 Call
 USA (Toll Free)
[+1 800 232 2078](#)

 Other Regions >

 Chat >

< Back



Menu



Did this guide **help you?**

Your feedback goes directly to our team.

It helps us improve future guides and products.


Takes **less than 10 seconds.**



Helpful



It's OK

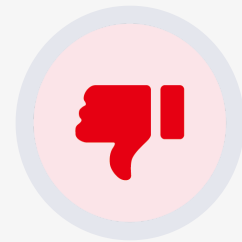

Need Help



Thank you for helping us improve.